

Diversey™

Accel® INTERVention® Wipes

One-Step Surface Cleaner and Disinfectant

Tuberculocidal, Virucidal, Bactericidal, Fungicidal, Broad-Spectrum Sanitizer

A ready-to-use pre-wetted disinfectant cleaner wipe based on proprietary AHP® technology to deliver fast, effective cleaning performance.

Features & Benefits

- Effective against key pathogens including MRSA, VRE, TB, Norovirus providing broad spectrum disinfection
- Kills Norovirus, Poliovirus and bloodborne pathogens and is Tuberculocidal all in 1 minute. It is also a 30-second broad-spectrum sanitizer
- Compatible with most hard, non-porous surfaces, while harnessing the power of AHP® technology. The AHP® active breaks down to oxygen and water after use
- No VOCs, no added fragrance and no NPEs. Non-irritating to eyes and skin
- Clean and disinfect high-touch surfaces in just 1 minute

Applications

- Clean and disinfect high-touch surfaces in just ONE minute
- Non-irritating to eyes and skin





Accel® INTERvention® Wipes

One-Step Surface Cleaner and Disinfectant

Use Directions: As a One-Step Cleaner Disinfectant*

- Pre-clean heavily soiled areas.
- Apply disposable wipe to hard, non-porous inanimate surfaces.
- All surfaces must remain visibly wet for 1 minute.
- Air Dry, wipe surfaces to dry and remove any residue, or rinse with potable water as necessary.

Prior to first use on a particular surface, it is recommended to test in a small, discreet area if there is concern of incompatibility, or contact a Diversey representative.

* Refer to product label for complete use instructions.

Technical data	Accel® INTERvention® Wipes
DIN	02448637
Color/Form	Colorless/Wipe
pH	3
Scent	Characteristic of ingredients
Shelf Life	3 years

Product	Pack size	Dilution	Product code	
Accel® INTERvention® Wipes	12 x 160 count Wipes 15.24 cm x 17.78 cm / 6 in. x 7 in. Canisters	Ready-to-use	100906585	🇨🇦
Accel® INTERvention® Wipes	12 x 60 count Wipes 25.4 cm x 25.4 cm / 10 in. x 10 in. Canisters	Ready-to-use	100906584	🇨🇦

Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at www.diversey.com or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate.

Accelerated Hydrogen Peroxide® and Design are registered trademarks of Virox Technologies Inc., used under license.